


## Qube Adherence Packaging

### SEALING INSTRUCTIONS FOR BLISTASSIST<sup>®</sup> TECHNOLOGY

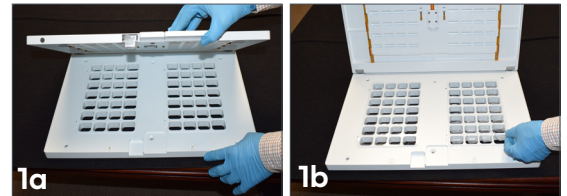

Open the cover on the BlistAssist<sup>®</sup> tray (1a) and place the blister into the tray. (1)




Fill the blisters with medication for each appropriate time pass using the BlistAssist technology, following instructions from McKesson Canada (2). Please note the following steps will be repeated for both blisters in the tray.




Apply the medication summary label to the inside cover of the card. Please note you will scan the barcode on this label using the BlistAssist integrated barcode scanner to verify placement of the card on the tray. (3)




Fold back the left grid panel of the Qube as shown. (4)

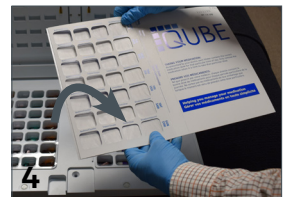


While keeping the grid panel in its folded position, flip over the card and remove the liner at a 45° angle. Ensure you avoid touching the adhesive panel during this process. (5)

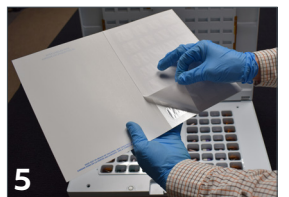

Flip the card back over and align the cut marks on the folded edge of the card with the corresponding guides on the BlistAssist tray (6a). Then, gently press the grid panel with your hand to engage adhesive. (6b)

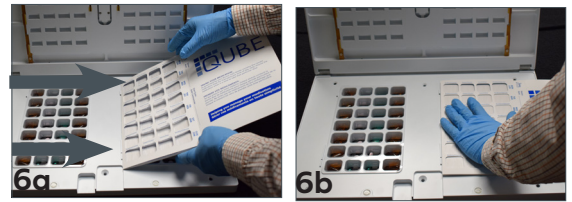


Using a roller, apply consistent pressure over the entire grid panel to ensure a tight seal. (7)

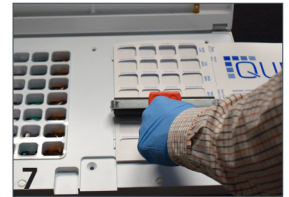


Remove the package from the tray, flip it over and fold the grid panel in the opposite direction and over the blister, ensuring it lays flat. (8)



Place the package face-down in a Qube manual sealing tray (9a). Using a roller apply firm, consistent pressure in a horizontal motion over the entire surface to ensure a solid seal. (9b)

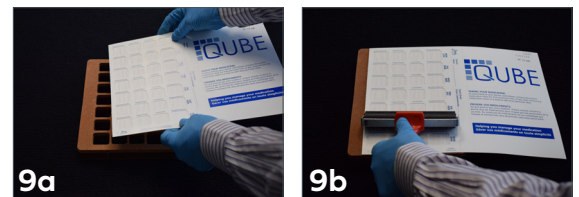

After the package is verified for accuracy, the Qube is now ready to dispense. (10)

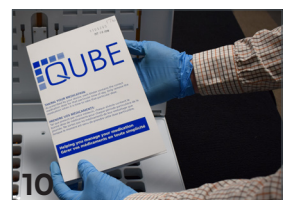

Before dispensing, the pharmacist should instruct the patient on the proper use of the Qube. To access the contents of the blister, the patient should puncture the label with their finger, peel back the label and remove the medication. The pharmacist must also advise the patient that the Qube is not child resistant. Keep out of the sight and reach of children and store in a cool, dry place, away from direct sunlight.

